




## Clean-Z Quick Guide | Assemble Unit


### Clean PAP Mask and Tubing

Insert the filter into the bag and connect the mask to the tubing. Place the mask into the bag. Close the bag around the tubing using the velcro. Attach the Clean-Z to the end of the tubing. The connector adapter might be needed based on your PAP equipment.

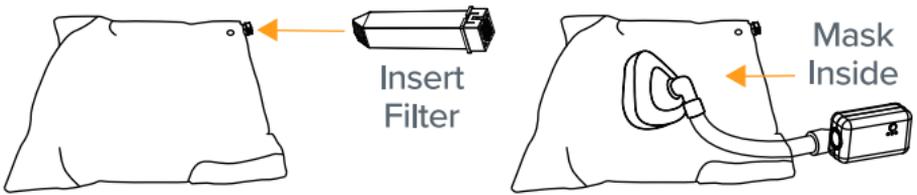


### Start Cleaning Process

Connect the power supply to the Clean-Z and plug into an electrical outlet. Press the power button. The Clean-Z will start and automatically run through a complete cleaning cycle.

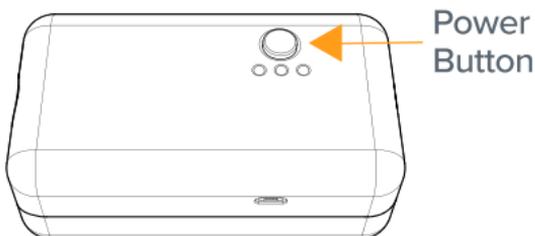

# Clean-Z Quick Guide | Use Unit

## Stages

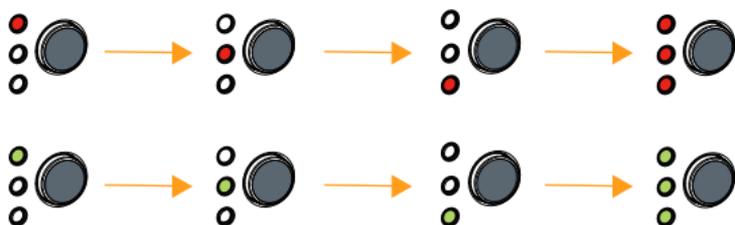

**Prep Stage:** The device will beep and display a slow flashing red light.

**Cleaning Stage:** The sequenced red light indicates ozone is being produced, sanitizing your device.

**Fresh Air Stage:** The device will now purge ozone from the PAP and show a sequence of green lights.

**Complete:** The sanitization is now complete. The device will beep and then show three solid green lights.

## Indications of Use

The Clean-Z is designed to sanitize an individual's Positive Airway Pressure (PAP) mask. Individuals who use a PAP machine for Obstructive Sleep Apnea (OSA) can use the Clean-Z to sanitize on a daily basis or when the user wants the added benefit of thorough sanitization.



See manual for a full list of warnings

**Warning:** Only use in a well-ventilated room. Allow the device to complete all cycles.

**Warning:** Keep away from all water sources. Keep away from flame or fire.

**Warning:** Keep away from children.

**Warning:** Do not disassemble or dismantle the Clean-Z device.

**Warning:** Do not run the Clean-Z for more than two consecutive cycles.

**Warning:** Do not use the Clean-Z while your PAP machine is in use. The Clean-Z is not a PAP machine.

For more information visit:

[www.motifclean-z.com](http://www.motifclean-z.com)

Manufactured for:  
Motif Medical LLC  
Dover, DE 19901