

Fitting

Swift FX Bella



- 1
- Hold the pillows at your nose, ensuring the ResMed logo is above the pillows facing up. Pull one strap around and behind your ear.
 - Ensure the strap lies flat and is not twisted.
 - Repeat with the other strap.



- 2
- Adjust the mask fit by pulling the upper straps along the notches until a comfortable fit is achieved. Repeat with the lower straps. Do not overtighten.



- 3
- Connect the free end of the short tube assembly to the device air tubing.

Swift FX for Her



- 1
- Holding the pillows at your nose, pull the headgear over your head.
 - Headgear should sit midway between the ears and eyes.



- 2
- Adjust the top strap first.
 - Pull the strap through the buckle one notch at a time.
 - Do not overtighten.



- 3
- Adjust the backstrap to fine-tune the fitting if necessary.
 - Do not overtighten.
 - The backstrap can be positioned over or under the hair.



- 4
- Swift FX pillows are designed to provide a natural fit.
 - Minor adjustment can help position the pillows at the most comfortable angle in the nose.
 - Connect the free end of the short tube assembly to the device air tubing.

Optional soft wraps

- Wrap each of the soft wraps around either side of the headgear, folding the top fastener over the lower one.
- The soft wraps should be positioned on the cheeks, with the fasteners on the outside.



S9™ Series
More. Comfort.



Combine a Swift FX for Her mask with ResMed's S9 AutoSet™ for Her therapy device for a comfortable, quiet, lifestyle-friendly system.

Swift™ FX Bella
NASAL PILLOWS SYSTEM

Less mask. More you.

RESMED

Nothing is like an FX mask.



Featuring a minimal design that provides a feeling of freedom, the **Swift™ FX Bella** is lighter, softer and more streamlined than ever.



Swift™ FX Bella
NASAL PILLOWS SYSTEM

Less mask. More you.

[ResMed.com/ChoicesForHer](https://www.ResMed.com/ChoicesForHer)



Swift™ FX Bella

Style and design with her in mind

The Swift FX Bella nasal pillows system combines a soft, lightweight design with a fit range specialized for women to meet their unique needs. The exclusive Bella loops accommodate a wide range of hairstyles, while the optional Swift FX for Her headgear offers another way to wear the mask. By offering more choices when it comes to customized fit and personal comfort, the Swift FX Bella lets her look and feel more like herself.



Bella loops

fit comfortably around the ears and stay out of her hair for a more streamlined look and feel

Dual-wall nasal pillows

provide a gentle, comfortable seal on contact

Soft, narrow cushion with integrated flexible chamber makes side sleeping comfortable and helps maintain seal during movement

Ultra smooth 360° rotating elbow

rotates with easy, fluid movement for flexibility in tube positions

Quiet vents

diffuse and direct airflow quietly and gently away from the user and bed partner


Spring-flex lightweight tube

stretches and bends easily for increased stability


"Lock and key" feature allows for easy attachment and detachment of Bella loops (or Swift FX headgear straps)

Size and name indicators make it easy to identify the size as well as the correct orientation of the mask

Flexible pillows with trampoline action accommodate movement in multiple directions while maintaining seal. Pillow sizes for her (XS, S and M) come packaged with mask to fit most female patients

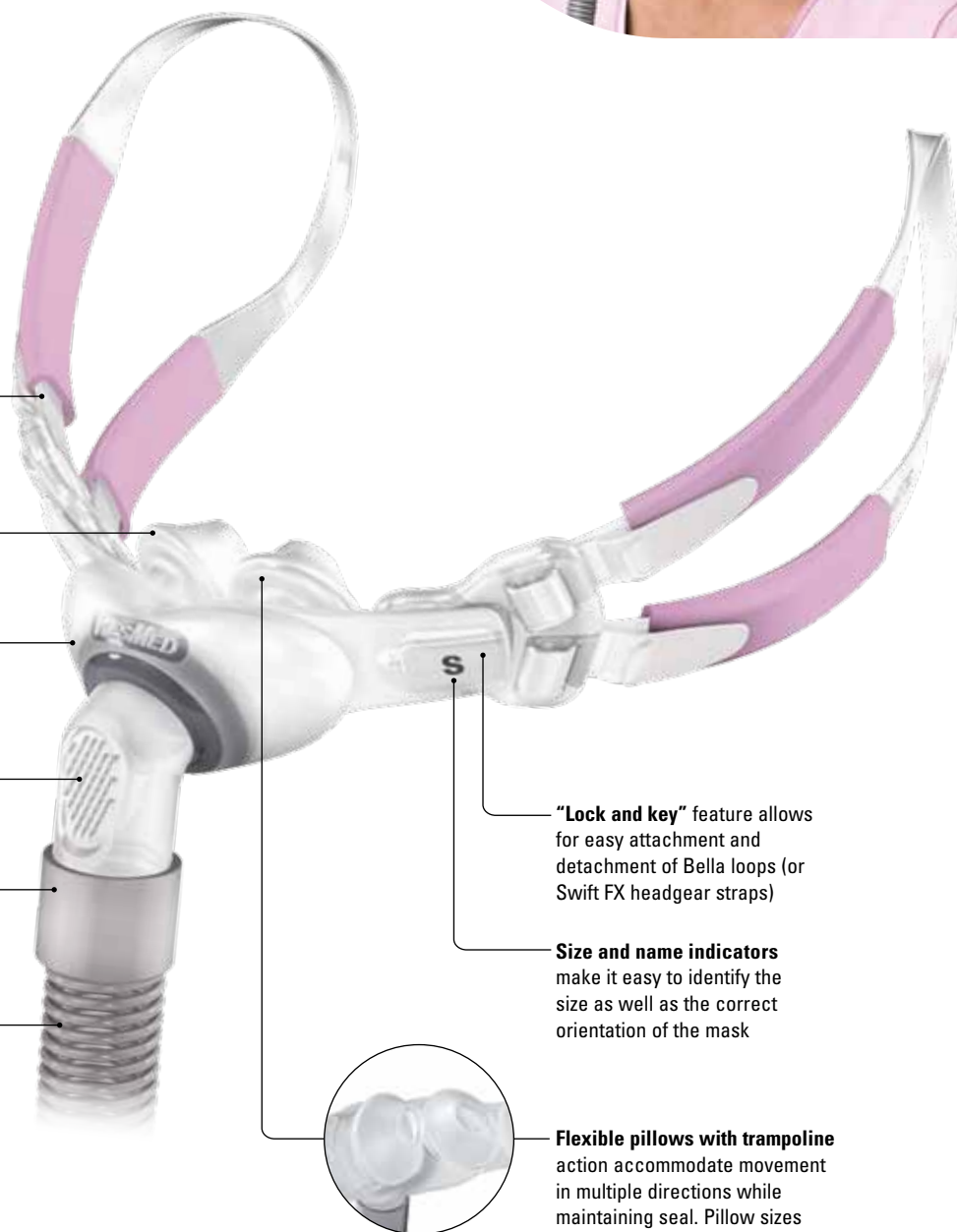

Swift FX Bella components

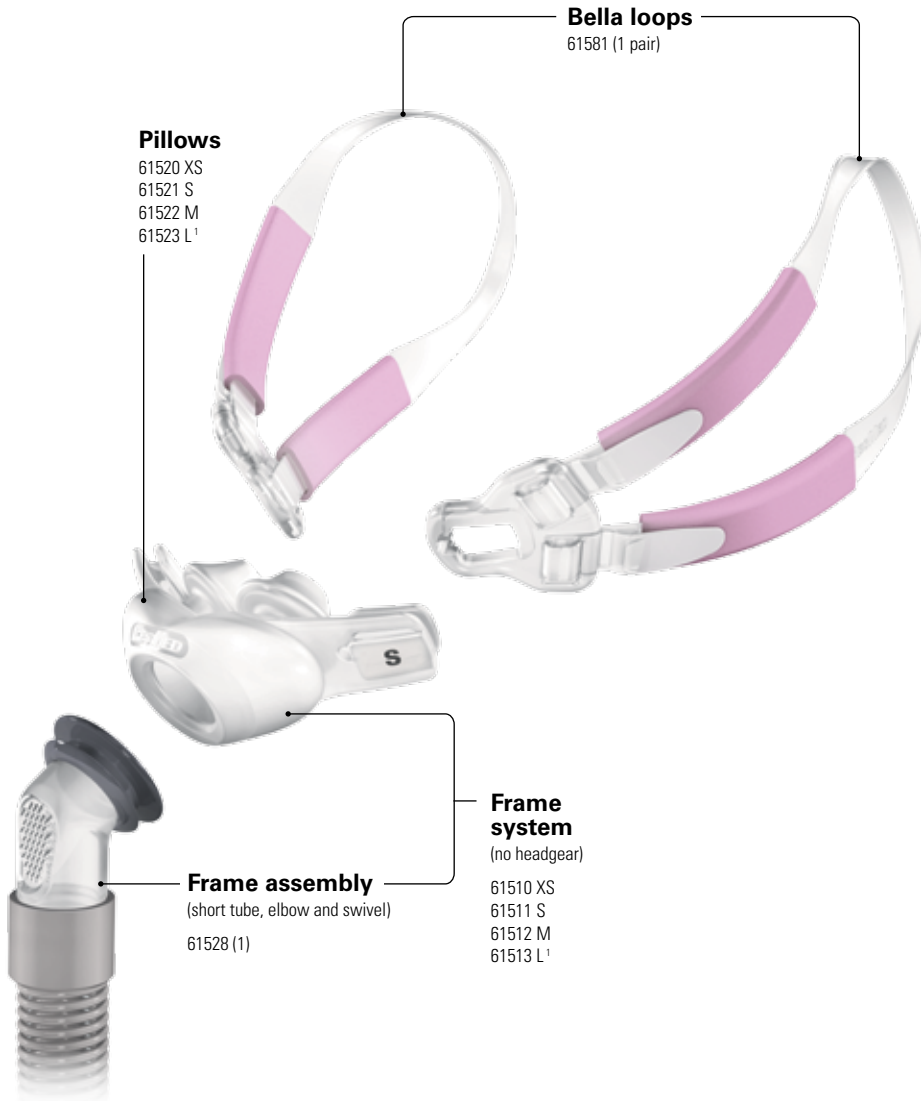

Pillows

- 61520 XS
- 61521 S
- 61522 M
- 61523 L¹

Bella loops

61581 (1 pair)

Frame system

(no headgear)

- 61510 XS
- 61511 S
- 61512 M
- 61513 L¹

Frame assembly

(short tube, elbow and swivel)

61528 (1)

PRODUCT CODES

Complete System

Swift FX Bella Nasal Pillows System

Style	Part Number
Swift FX Bella	61560


Swift FX Bella complete system includes Bella loops attached to the pillows frame, Swift FX for Her headgear with pink backstrap and pink soft wraps. Also included are Extra Small (XS), Small (S), and Medium (M) pillows.¹

Reimbursement Codes

HCPCS	Code Descriptor
A7034	Mask interface—mask or cannula, 1 per 3 months
A7033	Pillow for use on nasal cannula type interface—replacement, 2 pairs per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

Swift FX for Her headgear and Bella loops combo pack

61580

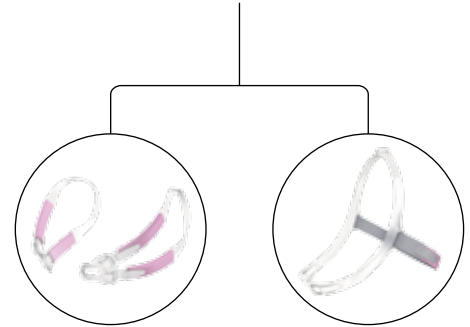

Bella loops

(1 pair)

61581

Swift FX for Her headgear assembly

(headgear, pink backstrap and pink soft wraps)

61543

Swift™ FX Bella
NASAL PILLOWS SYSTEM

Less mask. More you.

¹ 1 Large (L) pillows are available separately